


Fisher & Paykel Healthcare

Mask Family Seal Size Measurements

Nasal Masks



F&P Eson 2

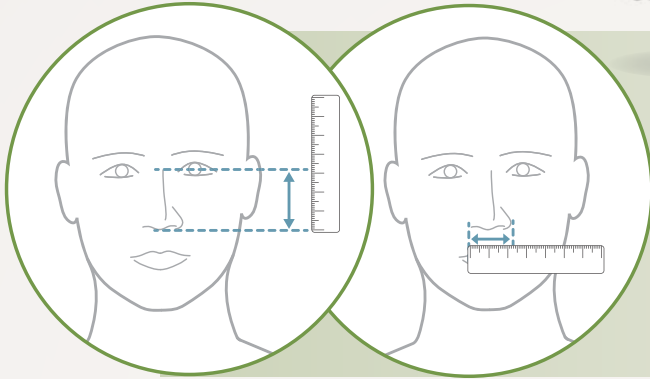

Nose Height

Nose Width

Small
Medium
Large


Nose height

Less than **4.4 cm (1.73 inches)**
Between **4.4 cm and 5.2 cm (1.73 inches and 2.05 inches)**
Greater than **5.2 cm (2.95 inches)**

Nose width

Less than **3.7 cm (1.46 inches)**
Between **3.7 cm and 4.1 cm (1.46 inches and 1.61 inches)**
Greater than **4.1 cm (1.61 inches)**

Full Face Masks

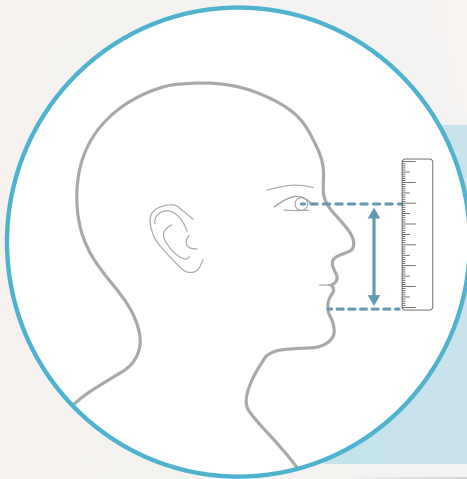

F&P Vitera



F&P Simplus

Small Less than **8.9 cm (3.5 inches)**
Medium Between **8.9 cm and 10 cm (3.5 inches and 3.94 inches)**
Large Greater than **10 cm (3.94 inches)**

Small Less than **6.6 cm (2.59 inches)**
Medium Between **6.6 cm and 7.5 cm (2.59 inches and 2.95 inches)**
Large Greater than **7.5 cm (2.95 inches)**

Measuring these points for sizing in a mirror may make the process easier for you.

These products may not be right for you. Read the warnings and instructions for use before purchase.

For further instructions on sizing, please refer to the Eson™ 2 and Simplus™ sizing guide REF 185048403 or Vitera™ sizing guide REF 612027

F&P, Eson, Vitera and Simplus are trademarks of Fisher & Paykel Healthcare Limited.
For patent information, see www.fphcare.com/ip

Prescription only. Use only as directed. Always follow the instructions for use.
Your healthcare professional will advise you whether this product is suitable for you/your condition.