



NASAL
YN-03




TECHNICAL SPECS

Service pressure range:	4-30 cm H ₂ O
Net Weight	99g
Connection tube:	22mm (diameter)
Flow Resistance (50LPM):	<0.5cm H ₂ O
Flow Resistance (100LPM):	<1.5cm H ₂ O
Pressure sound level:	25dB (A)
Dead space:	<142ml
Certification:	CE

MAIN FEATURES

- Adjustable Headgear with straps
- Soft silicon cushion
- Silent expiration system
- No forehead support
- Up to 30 cm H₂O
- Easy to handle clips (no magnetics)

PRESSURE-FLOW CURVE

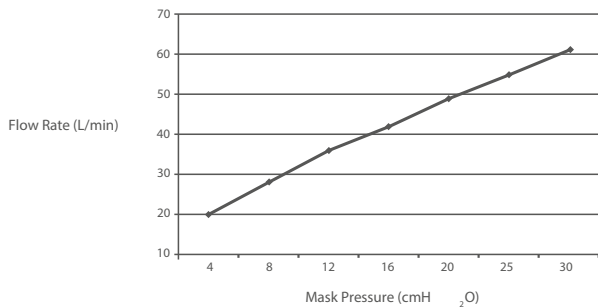


ORDERING INFORMATIONS

Part Number	Model	Size
MDA2MNM005	YN-03	S
MDA2MNM002	YN-03	M
MDA2MNM006	YN-03	L




NASAL
YN-03

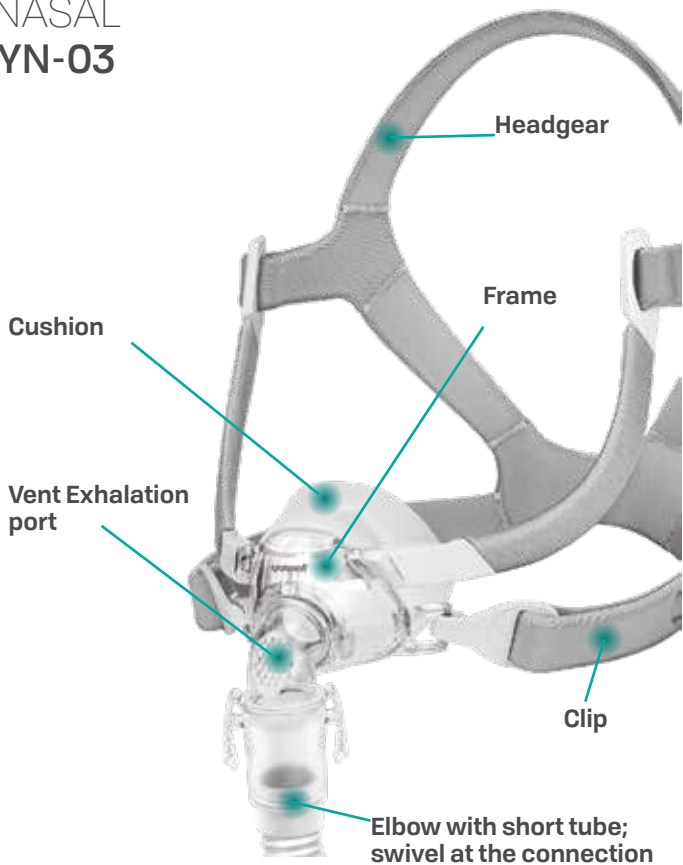

FITTING

Use a standard conical connector if pressure readings and / or supplemental oxygen are required.

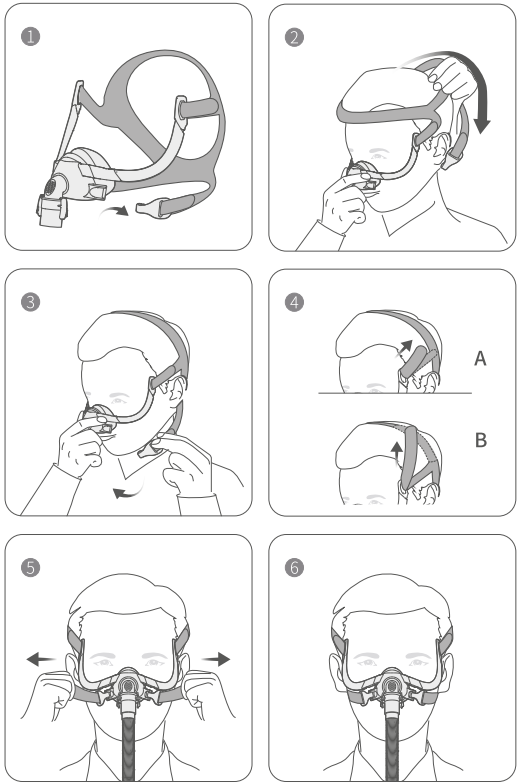

1. Unclip one of the lower headgear clips from the frame.
2. Hold the mask steadily on your face and pull the headgear over your head.
3. Bringing the lower straps below your ears, clip the headgear to the lower part of the frame.
4. Undo the fastening tabs on the upper headgear straps. Pull the straps evenly towards the top of head or above ears until the mask is stable and comfortable. Reattach the fastening tabs.
5. Undo the fastening tabs on the lower headgear straps. Pull the straps evenly until the mask is stable and comfortable. Reattach the fastening tabs.
6. Your mask and headgear should be positioned as shown in the illustration.

Note
Connect the device air tubing to the swivel of mask and ensure it is firmly connected.